



## **'EXTRA STRENGTH' CLEANER**

*FOR USE WITH RIGID GAS PERMEABLE SILICONE ACRYLATE, FLUOROSILICONE ACRYLATE CONTACT LENSES AND HARD CONTACT LENSES.*

### **DESCRIPTION:**

Optimum 'Extra Strength' Cleaner is a sterile, non-preserved aqueous solution of cocoamphodiacetate and glycols, and contains no suspended abrasive particles.

### **ACTIONS:**

Optimum 'Extra Strength' Cleaner eliminates surface deposits, grease, oils, lipids, salts, and cosmetic residue from silicone acrylate, fluorosilicone acrylate rigid gas permeable contact lenses and hard contact lenses.

### **INDICATIONS (USES):**

Use Optimum 'Extra Strength' Cleaner to clean silicone acrylate, fluorosilicone acrylate rigid gas permeable contact lenses and hard contact lenses.

### **CONTRAINDICATIONS**

#### **(REASONS NOT TO USE):**

If you are allergic to any ingredient that is used in Optimum 'Extra Strength' Cleaner, do not use this product.

### **WARNINGS:**

- Optimum 'Extra Strength' Cleaner contains no preservatives to which some people may be allergic, but see contraindications.
- Optimum 'Extra Strength' Cleaner is NOT for use directly in the eye.
- Serious damage to the eye and loss of vision may result from wearing contaminated or damaged contact lenses. Immediately contact your eye care practitioner to identify the cause of any unexplained eye discomfort, watering, vision change, redness of the eye, and to begin any necessary treatment.
- keep the bottle tightly closed when not in use.
- Never touch the tip of the bottle to any surface as this may contaminate the solution.
- Do not use with soft (hydrophilic) contact lenses.

### **PRECAUTIONS:**

Optimum 'Extra Strength' Cleaner is NOT for use directly in the eye. Rinse lenses thoroughly before placing in your lens case. If Optimum 'Extra Strength' Cleaner is accidentally placed in the eye, remove the lens and flush eyes with copious amounts of clean water. Optimum 'Extra Strength' Cleaner will sting and cause temporary redness if instilled in the eye.

- Always wash the hands with a mild soap and rinse well before handling your lenses. Never use soap containing oils or lanolin.
- Store solution at room temperature. Overheating or freezing may affect the solution.
- Keep out of reach of children.

- **CAUTION:** If cleaner is accidentally instilled in the eye, it will sting and cause redness. Should this occur, flush the eye repeatedly with clean water. If pain or redness persist, consult a physician.
- IF SWALLOWED:** Drink extra water. Cleaner contains cocoamphodiacetate and glycols.

## **ADVERSE REACTIONS**

### **(PROBLEMS AND WHAT TO DO):**

The following problems may rarely occur:

- Stinging, burning, or itching of the eyes (irritation).
- Excessive watering (tearing) of the eyes.
- Unusual eye secretions.
- Continued redness of the eye.
- Reduced sharpness of vision (visual acuity).
- Blurred vision.
- Sensitivity to light (photophobia).
- Dry eyes.

If you notice any of the above problems, remove and examine your lenses. If the problem stops and the lenses appear to be undamaged, clean, rinse, disinfect your lenses and re-insert them. If the problem continues, or a lens appears to be damaged, **IMMEDIATELY** remove the lens and consult your eye care practitioner. If any of the above problems occur, a serious condition may be present. Seek immediate professional identification of the problem and start treatment to avoid serious eye damage. See your *Instructions for Wearers* Booklet for your lenses for more information.

## **DIRECTIONS:**

Wash hands with a mild soap. Place the lens in the cupped palm of the hand. Apply 1 or 2 drops of Optimum 'Extra Strength' Cleaner to both surfaces of the lens. Massage the lens and work up a sudsing pool for 30 seconds. Or alternatively, apply 1 or 2 drops of Optimum 'Extra Strength' Cleaner to both surfaces of the lens and massage the lens between the fingers, or as directed by your eye care practitioner. Rinse thoroughly with running tap water. Be sure basin plug is in place.

## **CAUTION:**

### **DO NOT PUT CLEANER IN THE EYE**

***EACH CONTAINER IS TAMPER-EVIDENT SEALED.  
IF SEAL AROUND CAP IS MISSING OR BROKEN,  
DO NOT USE THIS PRODUCT.***

**Distributed by:**  
**Lobob Laboratories, Inc.**  
**San Jose, CA 95131**  
**[www.LobobLabs.com](http://www.LobobLabs.com)**

**Comments or questions:**  
**1 (800) 835-6262**  
**E-mail: [Info@LobobLabs.com](mailto:Info@LobobLabs.com)**

**Printed 2013**

**P/N 99226**